Stat Profile® Ultra SO₂ CalibratorSO₂ Kalibrator, Βαθμονομητής SO₂, Calibrator de SO₂, Etalon SO₂, Calibratore SO₂,
SO₂ キャリブレータ, Calibrator de SO₂, SO₂-kalibratorSO₂ CALIBRATOR 1 2

LOT			
CALIBRATOR 1	811015	2010-05	
CALIBRATOR 2	811016	2010-05	

Calibrator Value, Kalibratorwert, Τιμή βαθμονομητή, Valor del calibrator,
Valeur étalon, Valore calibratore, キャリブレータ値, Valor do calibrator, Kalibratorvärde

CALIBRATOR 1	%	50.9
CALIBRATOR 2	%	97.5

Product Description

An aqueous material for calibrating the oxygen saturation channel for use with Nova Biomedical analyzers ONLY. Formulated at two levels (Calibrator 1, Calibrator 2).

Intended Use

For in vitro diagnostic use for calibrating the SO₂ channel on the Stat Profile Ultra Analyzers.

Methodology

Refer to Stat Profile Ultra Analyzer Instructions For Use Manual for Methodology and Principles.

Composition

An aqueous electrolyte solution whose reflectance characteristics give a signal that is equivalent to a known oxygen saturation value in whole blood. Each ampoule contains a minimum volume of 1.0mL. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF: NCCLS DOCUMENT M29-T2).

Warnings and Cautions:

Intended for in vitro diagnostic use. Refer to the Stat Profile Ultra Analyzer Instructions for Use for complete directions. Follow standard practices required for handling laboratory reagents.

Storage

Store at 15-30°C; DO NOT FREEZE. Expiration date is printed on each ampoule.

Directions for use

Contents must be shaken well prior to use. Refer to analyzer Instructions for Use for complete directions. Verify that the lot number appearing on the Calibrator table is the same lot number indicated on the ampoule.

Limitations

The Oxygen Saturation External Calibrator is intended for use on Nova Biomedical Analyzers ONLY. The external calibrator is not suitable for use on any other manufacturers' analyzer.

Traceability of Standards

Analyses are traceable to tonometry.

Assigned Calibrator Values

The CALIBRATOR VALUE for each level was determined at Nova Biomedical by using multiple runs of each level of calibrator at 37°C on multiple analyzers.

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