REF 35150

IVD 15°C - 30°C CE

Stat Profile® Critical Care Xpress CO-Oximeter Calibrator Cartridge

Kalibrierkassette, Βύσμα πακέτο βαθμονομητή, Cartucho de calibrador, Cartouche de calibreurs, Cartuccia calibratore, Cartucho de calibrador, キャリブレータ・カートリッジ, 측정기 카트리지, 定标品盒

LOT 903329

2010-03

tHb CALIBRATOR		DIL ≥180mL	SOLN F ≥620mL	 WASTE CONTAINER W
Dye, Viscosity Enhancers, Preservatives in an Aqueous Salt Solution, Farbstoff, Viskositätsverbesserer, Konservierungsmittel in einer wässrigen Salzlösung. Χρωστική, ενισχυτικά (ζυδούς, συντηρητικά σε ένα υδατικό διάλυμα άλατος, Colorante, mejoradores de la viscosidad, conservantes en solución salina acuosa, Colorant, activateurs de viscosité, agents de conservation en solution saline aqueuse, Colorante, miglioratori della viscosità, conservanti in soluzione salina acquosa, Corante, intensificadores de viscosidade, conservantes em solução salina aquosa, 塩素水溶液に着色料 粘度増強剤、保存剤 염분 수용액 (Aqueous Salt Solution)의 염료, 점도 강화제, 방부제 盐水溶液中的染料、粘度增粘剂、防腐剂 増粘剤、防腐剤		Deionized Water, Surfactant, Buffers, Preservatives, Deionat, Tensid, Puffer, Konservierungsmittel Απιονισμένο νερό, επιφανειοδραστικό μέσο, ρυθμιστικά διαλύματα, συντηρητικά, Agua desionizada, surfactante, tampones, conservantes, Eau désionisée, agent de surface, tampons, agents de conservation, Acqua deionizzata, surfattante, tamponi, conservanti, Água desionizada, Surfactante, Tampões, Conservantes, 脱イオン水、界面活性剤、緩衝剤、保存剤 탈이온수, 계면 활성제, 버퍼, 방부제 去离子水、表面活性剤、缓冲剂、防腐剂	Deionized Water, Surfactant, Preservatives, Deionat, Tensid, Konservierungsmittel, Απιονισμένο νερό, επιφανειοδραστικό μέσο, συντηρητικά, Agua desionizada, surfactante, conservantes, Eau désionisée, agent de surface, agents de conservation, Acqua deionizzata, surfattante, conservanti, Água desionizada, Surfactante, Conservantes, 脱イオン水、界面活性剤、保存剤 탈이온수, 계면 활성제, 방부제 去离子水、表面活性剤、防腐剤	
Units	X̄			
g/dL	16.3			
g/L	163			
mmol/L	10.1			

Description
For use in calibrating the CO-Oximeter on Stat Profile Critical Care Xpress (CCX) Analyzer. The above table lists the constituents for the calibrator, flush and diluent solutions contained in the Calibrator Cartridge. Contains the following aqueous solutions within individual flexible bags housed in a cardboard box: two bags of flush solution, diluent solution, calibrator and a flexible waste bag. The exposed bag fittings are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer.

Intended Use
For the quantitative determination of total hemoglobin, oxyhemoglobin, carboxyhemoglobin, methemoglobin, and deoxyhemoglobin in human blood using the Stat Profile Critical Care Xpress CO-Oximeter Analyzer.

Methodology
Refer to Stat Profile Critical Care Xpress Analyzer Instructions For Use Manual for Test Methodologies and Principles.

Composition
Solution volumes are listed in the table above. Contains no constituents of human origin, however good laboratory practice should be followed during handling of these materials. (REF: NCCLS DOCUMENT M29-T2).

Warnings and Cautions
Intended for in vitro diagnostic use. Refer to the Stat Profile Critical Care Xpress Instructions For Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Calibrator Cartridges in accordance with local guidelines. Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage
Store at 15-30°C. Expiration date is printed on the front of the Cartridge.

Directions For Use
Refer to Stat Profile Critical Care Xpress Analyzer Instructions For Use Manual for complete directions for installation of the cartridge. Cartridge is ready for use.

Limitations
For use on Nova Biomedical Analyzers only.

If not stored at room temperature and out of direct sunlight, the solution may deteriorate. Indications of deterioration may be high or low out of range control values or a noticeable change in calibration or both. Cloudiness or particulate matter may be a sign of possible deterioration.

This product is warranted to perform as described in this product insert, and Nova Biomedical disclaims any implied warranty or merchantability or fitness for any other purpose, and in no event shall Nova Biomedical be liable for any consequential damages arising out of the aforesaid expressed warranty.

Traceability of Standards
Traceable to precise hemoglobin measurement using the reference cyanmethemoglobin method.

Reference
ICSH, Recommendations for Haemoglobinometry in Human Blood. British Journal of Haematology, 13 (suppl.) pp 71-75, 1967.

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